


Very easy to care for.

GMFCS
Levels III-V



I am your shower and toilet chair, and a real water fan!

With my many different sub-frames, I will accompany you in everything you do in the bathroom - whether during transfers, during use as a toilet chair or in the shower. My back angle can be precisely adjusted for you so that you can bend forwards or backwards. Because my different sub-frames make me highly flexible, I can adapt to any room and provide you with ideal support.

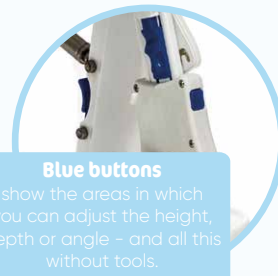

Adjustable seat angle at every height.





Holding bracket

In this way, the seat unit can be installed on any conventional toilet.

The bar can hold you permanently without restricting use.



Different sub-frames

for each spatial environment.



Mobile sub-frame



Fixed sub-frame



Bath sub-frame



Transfer and care sub-frame

Back angle

can be adjusted from $(-)\ 6^{\circ}$ to 18° .

Lateral pad

provides increased stability.

Seat pad

with two opening variations. Also available in an enclosed version.

Sub-frame

can be adjusted in height and angle.

Castors

facilitate uncomplicated transfer.

Headrest

can be adjusted in depth, height and angle.

Removable pads

can be cleaned easily.

Multifunctional seat unit

can be adjusted in seat depth.

Armrest

provides the necessary support.

Thigh guide

provides the necessary support.

Abduction wedge

guides the legs.

Footplate

diverse adjustments possible. Can be folded up for easy transfer.

Removable table

with padded stomach contact surface. This provides you with arm support and stimulation when going to the toilet.

**Toilet equipment**

You can use me as a toilet chair independently of a toilet.

**Travel base hts.**

Simply take me with you in the bag intended for the purpose.





Frame colour ☐ White

My tip:**Combine me with a shower WC by Closomat!**

Further information available on
www.closomat.com



closomat®
 sense the difference

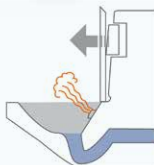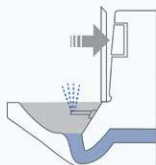

| Data | | Size 1 | Size 2 | Size 3 |
|--|-----------------------------|--|----------------------|----------------------|
| Width | Mobile sub-frames | 53 cm | 53 cm | 57 cm |
| | Stationary sub-frames | 51 cm | 51 cm | 53 cm |
| | Transfer and care sub-frame | 67 cm | 67 cm | 71 cm |
| Seat height | Mobile sub-frames | 39 – 55 cm | 39 – 55 cm | 44 – 58 cm |
| | Stationary sub-frames | 33 – 43 cm | 36 – 44 cm | 38 – 48 cm |
| | Transfer and care sub-frame | 51 – 78 cm | 51 – 78 cm | 51 – 78 cm |
| Seat width | | 20* / 25* / 30 cm | 25* / 30* / 36 cm | 33* / 38* / 43 cm |
| Seat depth | | 20 / 23 / 25 / 28 cm | 29 / 32 / 34 / 37 cm | 37 / 40 / 42 / 44 cm |
| Armrest height | | 13.5 cm | 17 cm | 20 cm |
| Back width | | 29 cm | 34 cm | 41 cm |
| Lower leg length, toilet seat | | 22 – 34 cm | 27 – 39 cm | 34 – 44 cm |
| Back height | | 42 cm | 50 cm | 62 cm |
| max. toilet height over which the product can be positioned | Mobile sub-frames | 52 cm | 52 cm | 52.5 cm |
| | Stationary sub-frames | 38 cm | 38 cm | 42 cm |
| Length | Mobile sub-frames | 74 cm | 74 cm | 81 cm |
| | Stationary sub-frames | 71 cm | 71 cm | 74 cm |
| | Transfer and care sub-frame | 84 cm | 84 cm | 87 cm |
| Front edge of hygiene opening to rear edge of mobile sub-frame | | 48 cm | 48 cm | 48 cm |
| Back angle | | (-)6°, 6°, 18° | (-)6°, 6°, 18° | (-)6°, 6°, 18° |
| Seat angle | | (-)5° – 25° | (-)5° – 25° | (-)5° – 25° |
| | | (-)15° – 15° | (-)15° – 15° | (-)15° – 15° |
| max. load | | 34 kg | 68 kg | 113 kg |
| HMV No. | | 33.40.01.4005 (use as a toilet attachment) / 33.40.04.1008 (use as a toilet chair) | | |

* can be adjusted using optional thigh guide



You can find more information on accessories & harnesses in the Price List.

